
THUASNE

Silistab® Epi



fr	Coudière anti-épicondylite	6
en	Anti-epicondylitis elbow brace	7
de	Ellenbogenstütze gegen Epicondylitis	8
nl	Anti-epicondylitis elleboogbrace	9
it	Gomitiera antiepicondilite	11
es	Codera antiepicondilitis	12
pt	Cotoveleira antiepicondilite	13
da	Albueunderlag mod tennisalbue	14
fi	Kyynärpäätuiki epikondyliitin (tenniskyynärpään) hoitoon	15
sv	Armbågsskydd mot epikondylit	17
el	Καλύπτρα αγκώνα κατά της επικονδυλίτιδας	18
cs	Loketní ortéza proti epikondylitidě	19
pl	Orteza stawu łokciowego przeciwko entezopatii nadkłykcia	20
lv	Antiepiķondilīta elkoņa pārsējs	22
lt	Alkūnės įtvaras epikondilitui gydyti	23
et	Epikondüliidi küünarnukitugi	24
sl	Komolčna blazinica proti epikondilitisu	25
sk	Antiepiķondylická bandáž	26
hu	Teniszkönyök elleni könyökrögzítő	28
bg	Налакътник срещу епикондилит	29
ro	Cotieră anti-epicondilită	30
ru	Налокотник против воспаления надмыщелка	31
hr	Anti-epikondilitis štitnik za laktove	33
zh	抗上髁炎护肘	34
ar	واقى مرفق مضاد لالتهاب اللقيمة	37

	1	19 - 22 cm
	2	22 - 25 cm
	3	25 - 28 cm
	4	28 - 31 cm
	5	31 - 34 cm
	6	34 - 37 cm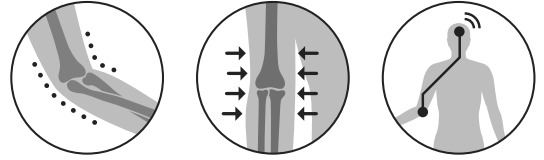

fr	Circonférence 8 cm sous le coude
en	Circumference 8 cm under the elbow
de	Umfang 8 cm unter dem Ellbogen
nl	Omtrek 8 cm onder de elleboog
it	Circonfrenza 8 cm sotto il gomito
es	Circunferencia de 8 cm bajo el codo
pt	Circunferência 8 cm abaixo do cotovelo
da	Omkreds 8 cm under albuen
fi	Ympärysmitta 8 cm kyynärpään alapuolelta
sv	Omkrets 8 cm under armbågen
el	Περίμετρος 8 εκ. κάτω από τον αγκώνα
cs	Obvod 8 cm pod loktem
pl	Obwód 8 cm poniżej łokcia
lv	Apkārtnērs 8 cm zem elkoņa
lt	Apimtis 8 cm žemiau alkūnės
et	Küünarnuki all ümbermõõt 8 cm
sl	Obseg 8 cm pod komolcem
sk	Obvod 8 cm pod laktom
hu	Kerület könyök alatt 8 cm-rel
bg	Обиколка 8 cm под лакътя
ro	Circumferință de 8 cm sub cot
ru	Окружность 8 см ниже локтя
hr	Opseg 8 cm ispod lakta
zh	肘部下周長8厘米
ar	محيط بمقدار 8 سم أسفل المرفق

Maintien	Compression	Effet proprioceptif
Support	Compression	Proprioceptive effect
Halt	Kompression	Propriozeptive Wirkung
Fixatie	Compressie	Proprioceptieve werking
Sostegno	Compressione	Effetto propriocettivo
Sujeción	Compresión	Efecto propioceptivo
Manutenção	Compressão	Efeito proprioctivo
Støtte	Kompression	Proprioceptisk effekt
Tuki	Kompressio	Liikeaistiin kohdistuva vaikutus
Stöd	Kompression	Proprioceptiv effekt
Συγκράτηση	Συμπίεση	Αποτελεσμα ιδιοαετικότητας
Stabilizace	Komprese	Proprioceptivní účinek
Podtrzymywanie	Kompresja	Propriocepcja
Atbalsts	Kompresija	Proprioceptīvs efekts
Palaiķymas	Kompresija	Propriocepčinis poveikis
Hooldus	Kompressioon	Proprioceptiivne toime
Oprijem	Kompresija	Proprioceptivni učinak
Podpora	Kompresia	Proprioceptivny účinok
Támasz	Szorítás	Proprioceptív hatás
Поддръжка	Компресия	Проприоцептивен ефект
Sustinere	Compresie	Efect proprioceptiv
Стабилизация	Компрессия	Проприоцептивный эффект
Podpora	Kompresija	Učinak proprioceptivnog vježbanja
穩定	压紧	本体感
الدعم	الضغط	مفعول استقبال الحس العميق

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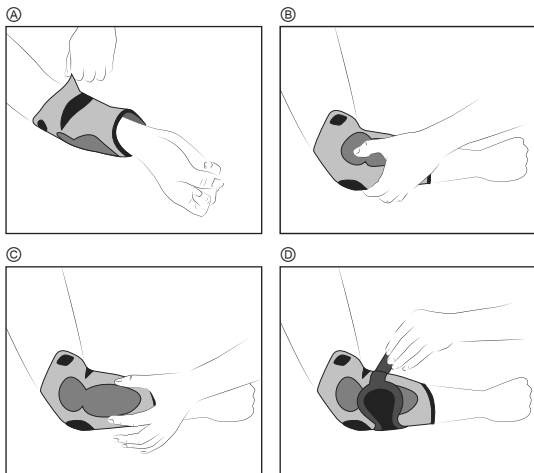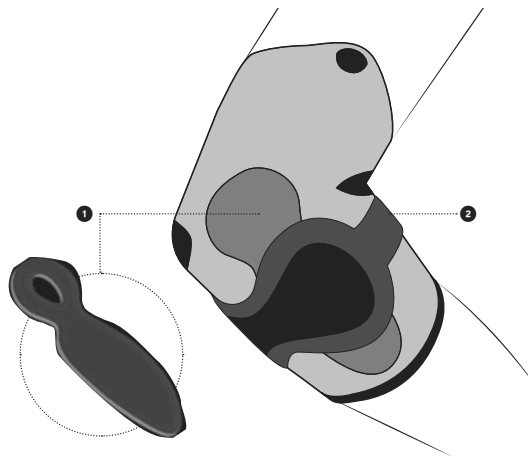ANTI-EPICONDYLITIS ELBOW BRACE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.
This device is an elastic proprioceptive elbow brace providing targeted pressure to relieve tendon pain.

Composition

Polyamide - elastan - polyester - polyurethane - silicone

Properties/Mode of action

- Elbow support with anatomically-shaped inserts. ①
- Anatomically-shaped knit for a good fit.
- Removable strap allows pressure adjustment. ②

Indications

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).
Prevention of elbow tendinopathy recurrence.
Support for physical activity resumption.

Contraindications

- Do not use the product if the diagnosis has not been confirmed.
- Do not use in the event of known allergy to any of the components.
- Do not apply the product in direct contact with broken skin.

Precautions

- Verify the product's integrity before every use.
- Do not use the device if it is damaged.
- Choose the appropriate size to fit the patient, referring to the size chart.
- It is recommended that a healthcare professional supervises the first application.
- Strictly comply with your healthcare professional's prescription and recommendations for use.
- For hygiene, security and performance reasons, do not re-use the product for another patient.
- Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).
- It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.
- In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.
- If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.