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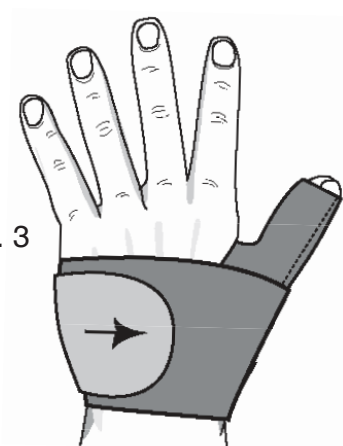
www.ezywrap.com

PROFESSIONAL PRODUCTS, INC.
 54 HUGH ADAMS ROAD
 DEFUNIAK SPRINGS, FL 32435
 PROFESSIONAL PRODUCTS, INC.
 COPYRIGHT © 2017

FABRICANT :

D-30175 Hannover Germany
 Schiffgraben 41
 MOSS GmbH
 EUROPEAN AUTHORIZED REPRESENTATIVE:

C / A
 ID # *274427*
 R348

04 74 60 75 03
 04 74 60 71 06
 commercial@sober.fr

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USA & CANADA (800) 234-8004
 INTERNATIONAL (859) 882-5731
 (800) 274-8006

DEFUNIAK SPRINGS, FL 32435
 PROFESSIONAL PRODUCTS, INC.

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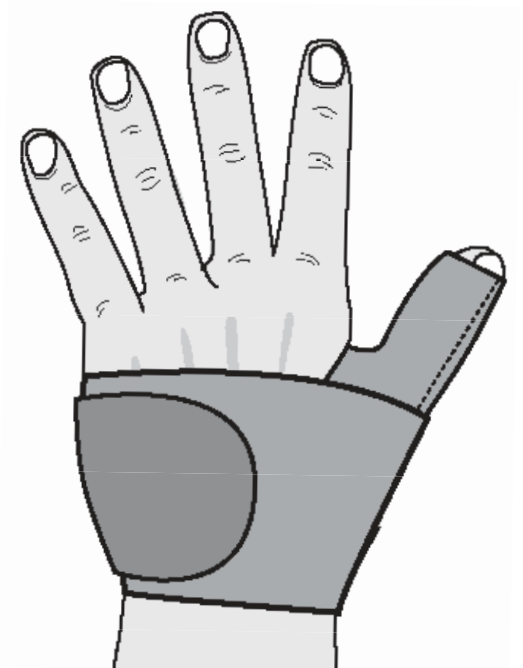
FOR OVER 50 YEARS *Proudly* **MADE IN USA** 

English APPLICATION INSTRUCTIONS

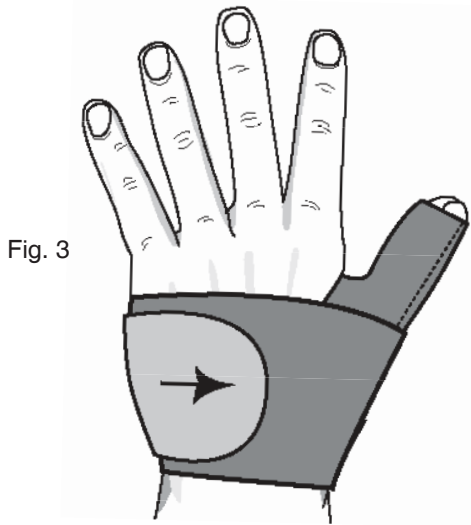
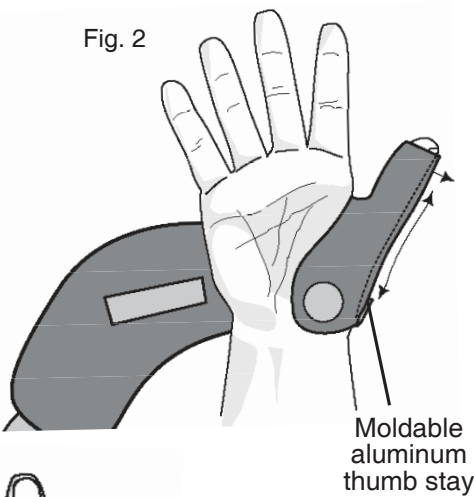
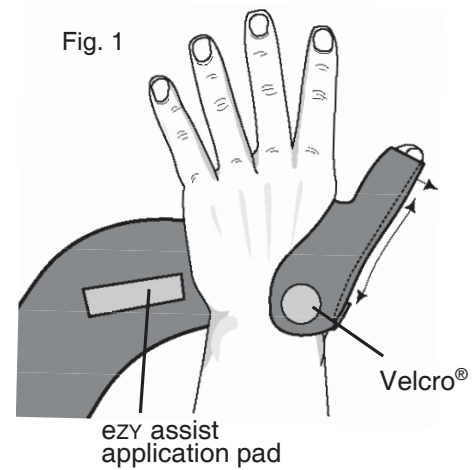
INDICATIONS

Ideal for strains, sprains, the treatment of Carpal Tunnel Syndrome or thumb injuries.

Thoroughly read the following information prior to application.
Product function requires proper application.

A Insert thumb into contoured thumbhole. Contour shape optional moldable aluminum thumb stay as desired. (Fig. 1 left), (Fig 2 right)

B Complete wrapping. Wrap compression strap around wrist. secure ezy assist application pad to Velcro®. Adjust compression strap until it feels comfortable and secure (Fig. 3).

CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. Reshape, dry flat

WARNING: INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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