Malling Perching Stools

VG833

VG834

VG835

VG836

VG837

VG837P

VG837B

VG837BR

Fixing and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

INTRODUCTION

Thank you for deciding to purchase the Malling Perching Stool from Aidapt. The Malling Perching Stool range is manufactured from the finest material and components available, when installed and used correctly it is designed to give many years of reliable trouble-free service. The Malling Perching Stool is available with optional arms, back or padded back.

Malling Perching Stools are delivered complete and do not require assembly.

N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user.

VG833	Malling Perching Stool (Plain)
VG834	Malling Perching Stool (Arms Only)
VG835	Malling Perching Stool (Arms & Plain Back)
VG836	Malling Perching Stool (Plain Back Only)
VG837	Malling Perching Stool (Arms & Padded Back)
VG837P	Malling Perching Stool (Padded Arms & Padded Back)
VG837B	Black Malling Perching Stool (Arms & Padded Back)
VG837BR	Brown Malling Perching Stool (Arms & Plain Back)

159 kg (25 stone) weight limit.

Do not exceed the weight limit stated – doing so could damage the equipment or put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

USE OF THE PERCHING STOOL

When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.

Do not allow children or unauthorised persons to play with or use the stool.

Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.

Do not use the perching outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.

Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.


HEIGHT ADJUSTMENT

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and that it fits neatly and snugly around the outside leg. (see fig. 2)

NB. The perching stool is designed to slant to the front. Do not adjust any of the legs to make the seat level as this will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool, including fracture of the tubular and other sections.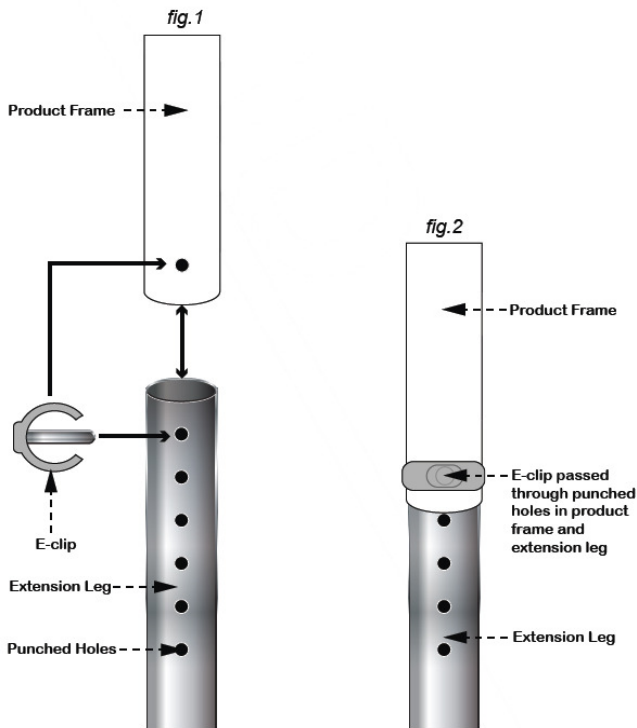

COMPOSITION

The perching stools are made from mild steel PPA/ epoxy coated with upholstered CMHR foam and fire retardant vinyl seats or contoured polyurethane seats.

CLEANING

Clean your product using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. After cleaning, wipe with a dry, soft cloth if necessary. Do not leave in direct contact with a heat source.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/bolts/castors and other screw in/bolt in/push in components.
- Also check all upholstery (if applicable) for security, splits, exposed staples, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)